moveable

movido explore & play

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Explanation of the symbols

EN

- WARNING** Warning of accident and injury hazards
CAUTION! Warning of possible technical damage.

Intended use

The ***movido explore*** (CK01) and ***movido play*** (CK02) are medical devices for use as a knee in a prosthetic leg for individuals with lower limb amputation or congenital dysmelia, specifically for knee disarticulation, transfemoral amputation and hip disarticulation.

Indication

The ***movido explore*** is suitable for children up to 25 kg of body weight and up to an age of approx. 6 years, the ***movido play*** up to 45 kg of body weight and up to an age of approx. 12 years.

Contraindication

If the child does not have the necessary movement or strength to use the knee safely. In the case of intensive sporting activity, a device designed for this purpose is recommended.

Function

- Stable and lightweight multi-axis children's knee
- Hydraulic swing phase control, adjustable to the child's activity.

- Low system height of approx. 36 mm (*movido explore*) and large flexion angle of 145°.
- Natural gait pattern and better adaptation to changes in pace.

Safety instructions

CAUTION! The penetration of dirt, sand, dust and moisture into the knee mechanism can cause premature loss of function and locking of the knee joint.

Alignment and assembly – figure ①

- Provide sufficient socket flexion for flexion contracture
- Consider the center of the foot as the alignment line
- Position trochanter approx. 15mm behind this line
- Upper anterior axis approx. 15 mm behind this line
- Attach the upper adapter according to the manufacturer's instructions
- Insert the 22 mm prosthesis tube completely and tighten with a torque of 10 Nm

CAUTION! Leave at least 10 mm space underneath the hydraulics for the piston rod, which protrudes slightly at maximum bending! – figure ②

CAUTION! Replace the rear screw of the upper pyramid adapter if it protrudes from the adapter and contacts the knee. – figure ③

Adjustment of the swing phase – figure ④

Use a 3 mm Allen wrench to remove the tube from the distal end of the knee joint.

The setting influences both flexion and extension

Turn to the right → Higher resistance

Turn to the left → Lower resistance

Make sure the knee is fully extended with each step.

Mounting knee cap and dust cover – figure ⑤

To protect the knee joint and the user's clothing, a knee protection cap and a dust cover are supplied with each knee joint.

The knee protection cap can be easily fitted to the knee joint. If required, this cap can be fixed to the front bar with two-component adhesive.

CAUTION! The adhesive may only be applied to both sides of the front bar.

First pull the dust cover onto the dismantled lower leg tube. Then fit and secure the tube as specified. Then slide the sleeve over the knee joint. Align the fixation eye anteriorly. Replace the front screw of the pyramid attachment with the fixing screw supplied, apply Loctite to it and tighten to 5 Nm.

CAUTION! If the knee cap and dust cover are not used, the warranty will be void.

Problems

The movido does not automatically come into extension when I bend the knee in my hand.	The movido regulates the swing phase hydraulically. By hand, the movido therefore does not always fully extend, but when walking, the mass of the foot and shoe is sufficient to extend the knee.
The movido does not come fully into extension when walking.	Reduce the resistance of the hydraulics by turning the adjusting screw on the underside of the hydraulics to the left. (Allen wrench size 3 mm)
The movido flexes too far when walking.	Increase the resistance of the hydraulics by turning the adjustment screw on the bottom of the hydraulics clockwise.
The movido buckles when walking	Check the alignment. Ensure sufficient socket flexion in the case of a flexion contracture.

Maintenance

Check the knee regularly, at least every 6 months, for function and defects. Replace the extension stop if necessary. The knee can be cleaned with water or a mild cleaning agent. Dust etc. can be removed with compressed air.

WARNING:

After an unplanned high load event, the prosthetic knee must be carefully checked. If you have any doubts as to whether the mechanics of the knee are still fully intact, contact the manufacturer. If you have any further questions about service, please contact the manufacturer.

Lifetime

The **movido** knees are tested in accordance with ISO 10328 for a weight up to 25/45 kg and 3,000,000 steps. The maximum service life is 3 years. Use beyond the service lifetime may result in unexpected defects and risks to the user.

Technical information

Knee model	<i>movido explore</i>	<i>movido play</i>
Weight	190 g	250 g
Build Height	102 mm	113 mm
System Height	36 mm	49 mm
Upper connection	Pediatric pyramid	Pediatric pyramid

Knee model	<i>movido explore</i>	<i>movido play</i>
Lower connection	Tube 22 mm	Tube 22 mm
Max. body weight	25 kg	45 kg
Temperature Range normal use		-10 - 40°C 32 - 104°F
Temperature range storage		-10 - 60°C 14 - 140°F

Scope of delivery

- ***movido explore* or *play***
- Knee cap
- IFU
- Dust Cover

Recycling

You can return the product to the manufacturer at the end of its life. The manufacturer will take care of responsible disposal.

If you dispose of this product in any other way, be sure to comply with local laws and regulations regarding disposal.

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