RS5 Silhouette® LP

DE Gebrauchsanweisung	3
EN Instructions for use	6
FR Instructions d'utilisation.....	9
NL Gebruiksaanwijzing	12

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Product Number: RS5

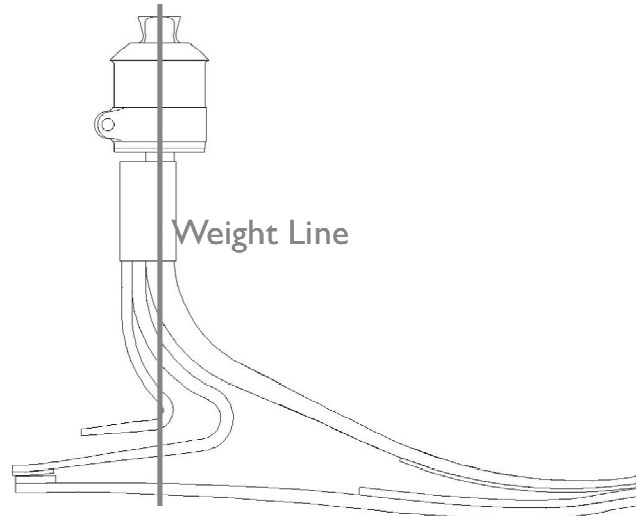
Assembly

The Silhouette® LP foot module is pre-assembled consisting of a graphite component (keel), sole plate, Spectra™ sock, tube clamp, and foot shell. The Silhouette® LP requires a male or female tube clamp to connect to other prosthetic components. When assembling the tube clamp, ensure the pylon and insert are fully seated before tightening. Always attach the tube clamp with the bolt in the posterior position. If height adjustments are required after the pylon is cut, the optional Versa-Clamp (AC-123L) should be used with the appropriate height spacer ring(s). Foam *stiffening bumpers* for increasing heel stiffness are provided. After dynamic alignment, torque pyramid adjustment screws to the manufacturer's specifications. Secure pyramid adjustment screws with a thread locking adhesive (i.e., Loctite 242).

Bench Alignment

Prior to donning the prosthesis:

- Plantarflex/Dorsiflexion foot module to match the shoe heel height.
- Adduct/Abduct socket to provide appropriate frontal plane angle.
- Flex/Extend socket to provide appropriate sagittal plane angle.
- Move the socket linearly to ensure the weight line falls along the anterior edge of the pylon (see illustration).

Dynamic Alignment

During *loading response*, the sole plate and heel lever store energy and release it during *midstance*. This action provides momentum for the keel to store energy and release it during *terminal stance*. To optimize the heel to toe rollover motion, adjust the following variables:

- Anterior/posterior foot placement
- Dorsiflexion/plantarflexion
- Heel stiffness

Troubleshooting

Heel too soft

Symptoms

- Foot flat occurs too rapidly
- Toe feels excessively stiff
- Knee hyperextension

Solutions

- Shift socket anteriorly in relation to the foot
- Attach foam *stiffening bumpers*

Heel too hard	<p><i>Symptoms</i></p> <ul style="list-style-type: none"> • Rapid knee flexion, instability • Heel to toe progression to rapid • Lack of energy return sensation <p><i>Solutions</i></p> <ul style="list-style-type: none"> • Shift socket posteriorly in relation to the foot • Verify appropriate foot module category
Foot module too stiff	<p><i>Symptoms</i></p> <ul style="list-style-type: none"> • Flat spot in rollover motion at slow cadences <p><i>Solutions</i></p> <ul style="list-style-type: none"> • Consider a lower category foot module
Foot module too soft	<p><i>Symptoms</i></p> <ul style="list-style-type: none"> • Clicking noise at <i>initial contact</i> • Excessive toe deflection with high impact activity <p><i>Solutions</i></p> <ul style="list-style-type: none"> • Consider a higher category foot module

Stiffening Bumpers Foam *stiffening bumpers* are included to adjust the heel stiffness during *loading response*. The bumpers may be temporarily attached with tape between the heel levers. If the bumpers provide excessive heel stiffness, exclude or trim as appropriate. For permanent placement, adhere bumpers using Barge or similar contact cement.

Spectra™ Sock A Spectra™ sock is provided to protect the foot shell and minimize noise. Spectra™ socks must be replaced at intervals appropriate to the user’s activity level. Failure to inspect and replace the Spectra™ socks may prematurely wear the foot module, and will void the warranty.

Foot Shell When removing or installing the foot shell, use the Foot Shell Removal Tool (ACC-00-10300-00) to prevent damage to the foot module.

Silhouette® LP
 Minimum clearance: 168mm-191mm
 Maximum user weight: 166 kg (365 lbs)
 Available sizes: 22cm-31cm
 Heel height: 10mm

Maintenance The foot module requires periodic maintenance.

- Inspect the foot module every six months. If the user is more active, more frequent inspection may necessary. Service as necessary. Replace Spectra™ sock and/or foot shell if worn to prevent damage to the graphite components.
- The foot module may be cleaned and/or disinfected with soap and warm water. If the foot is exposed to water, remove the foot shell to clean and disinfect its interior.

Warnings Failure to adhere to the guidelines of the *Instructions for Use* will void the warranty.

- Only use the provided Tube Clamp Insert, Silhouette/VS (ACC-00-12400-00). The Tube Clamp Insert, Silhouette/VS is not interchangeable with other Silhouette foot products as it is machined to match an individual Silhouette foot product. Failure to comply may prevent sufficient tube clamp force, cause product failure, and/or cause user injury. A replacement Tube Clamp Insert, Silhouette/VS may be ordered by providing Customer Service with the Silhouette serial number.
- Never use the foot module without a foot shell. Failure to comply may cause premature wear, loss of function, and/or product failure.
- Always use the foot module with a sock and shoe. Failure to comply may cause premature wear, loss of function, and/or product failure.

- Never allow aggregates such as sand to remain in the foot shell. Upon exposure to aggregates, immediately disassemble foot module and rinse with water. The abrasive properties of aggregates will quickly wear the graphite components of the foot module.
- Never assemble the Silhouette® LP foot module with the tube clamp insert extending past the end of the clamp.
- Failure to install the tube clamp with the bolt in the posterior position may result in failure and/or bodily injuries.
- Never assemble the *sole plate* inside the Spectra™ sock of the Silhouette® LP foot module. This will lead to premature failure of the foot module.
- Never use the Silhouette® LP foot module without the *sole plate*. *Sole plate* removal will adversely affect the performance of the foot module. The *sole plate* fills the mid-foot void between the heel lever and toe lever, preventing potential user injury.
- Freedom Innovations foot modules are manufactured to fit industry standard pyramids and receivers. It is the prosthetist's responsibility to select and/or fabricate properly fitting attachment components.
- Discontinue use and consult your prosthetist if any part of the prosthesis starts to make noise.
- Inform your prosthetist if you lose or gain a significant amount of weight.
- Freedom Innovations foot products are manufactured and tested for a particular weight and activity impact level. Use by another user for whom it was not originally manufactured may cause injury and shall void any written or implied warranty.

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