

# Taleo Side Flex 1C58

Ready for everyday life.

Taleo Side Flex was designed for active individuals who navigate varied indoor and outdoor environments and place a high value on effortless walking and socket comfort.

**Designed for optimal outcomes:**

- Terrain Adaptation Unit provides mediolateral adaptability of 20° (10° medial and 10° lateral) for immediate and full-surface ground contact while walking and standing, including on uneven surfaces and slopes
- The curved bottom of the Terrain Adaptation Unit enables more controlled forward movement during rollover than with low profile feet with a conventional foot adapter
- For users with limited clearance
- Three different heel wedges to optimize gait dynamics and reduce impact forces
- Drainage holes in footshell prevent water from collecting in the interior of prosthesis
- Protected against fresh, salt and chlorine water



More than a foot.  
A foundation.

**Technical Data**

K-Level	K3
Max. body weight	287 lbs / 130 kg
Side	left (L), right (R)
Sizes	22 - 30 cm
Weight without footshell*	20.49 oz / 581 g
Footshell shape	Slim shape (S) with 15 ± 5 mm heel height (size 22-25 cm)
	Normal shape (N) with 10 ± 5 mm heel height (size 24-30 cm)
Footshell color	beige (4), light brown (15)
Weight with normal footshell*	28.39 oz / 805 g
Build height	3 13/16 in / 97 mm (with normal footshell*)

\* Reference size is 26 cm.



# Taleo Side Flex 1C58



Order form

Customer		Shipping address (if different from customer address)	
Customer no.	<input type="text"/>	Customer no.	<input type="text"/>
Company	<input type="text"/>	Company	<input type="text"/>
Street	<input type="text"/>	Street	<input type="text"/>
Postal code/city	<input type="text"/>	Postal code/city	<input type="text"/>
O&P professional	<input type="text"/>	PO no.	<input type="text"/>

## Selection of the spring stiffness relative to

### 1 body weight and activity as well as

Body weight lbs [kg]	Normal activity level	High activity level
Up to 113 [51]	1	2
114 - 129 [52 - 58]	2	3
130 - 149 [59 - 67]	3	4
150 - 170 [68 - 77]	4	5
171 - 195 [78 - 88]	5	6
196 - 220 [89 - 100]	6	7
221 - 254 [101 - 115]	7	8
255 - 287 [116 - 130]	8	-

### 2 foot size

Foot size	Stiffness							
	1	2	3	4	5	6	7	8
22						-	-	-
23							-	-
24							-	-
25								-
26								**
27	-	-						**
28	-	-						**
29	-	-	-				**	**
30	-	-	-				**	**

Slim footshell available (15 ± 5 mm heel height)
  Both footshells available
  Normal footshell available (10 ± 5 mm heel height)

\*\* Do not combine this configuration with a 3C88-3 or 3C98-3 C-Leg 4.







### Example for ordering 1C58=R26-4-P/4N

Article no.	Side	Size	Stiffness	P /	Color	Shape
1C58=				P /		
	Side	Size [cm]	Color		Shape	
	Right R	22, 23, ..., 30	Beige 4		Slim S	
	Left L		Light brown 15		Normal N	
					22-25 cm	
					24-30 cm	

The scope of delivery includes the 1C58 foot module, 2C15 footshell including connection cover, 2F50 heel wedge set (comprising three different degrees of hardness) and a black Spectra sock.



### Recommended combinations\*\*\*

TT	TF
 <p>4R220* DVS + 6Y94=*-F DVS liner and 453A30 / 453A40 ProFlex Plus knee sleeve</p>	 <p>7T451=1 SiOCX TF Pro; 6Y110 Skeo Sealing liner and 21Y21 ClickValve</p>
 <p>4R10=111 Quickchange adapter</p>	 <p>3C88-3* / 3C98-3* C-Leg 4 3B1-3* Genium 3B5-3* Genium X3</p>
	 <p>4R57=WR Waterproof rotation adapter</p>
	 <p>3F1=1 + 99B120=* Functional Cosmesis for C-Leg 4 3F1=2 + 99B120=* Functional Cosmesis for Genium</p>

\*\*\* All components are sold separately and are available Ottobock products that are compatible with the 1C58 Taleo Side Flex foot, which helps ensure optimal performance. Practitioners need to select components based upon individual patient criteria.